My Perfect Colon[®]

Natural Hydro-Therapy at Home

- EN User guide*
- IT Manuale d'uso*
- FR Manuel d'utilisation*
- DE Bedienungsanleitung*
- ES Manual de uso*
- PT Manual de utilização*
- PL Instrukcja obsługi*
- RU Руководство Пользователя*
- UA Інструкція Для Користувача*
- RO Manual De Utilizare*

*For "Basic" & "Travel" version

US
Patent





Italian
Patent

Other countries
Patent pending

www.myperfectcolon.com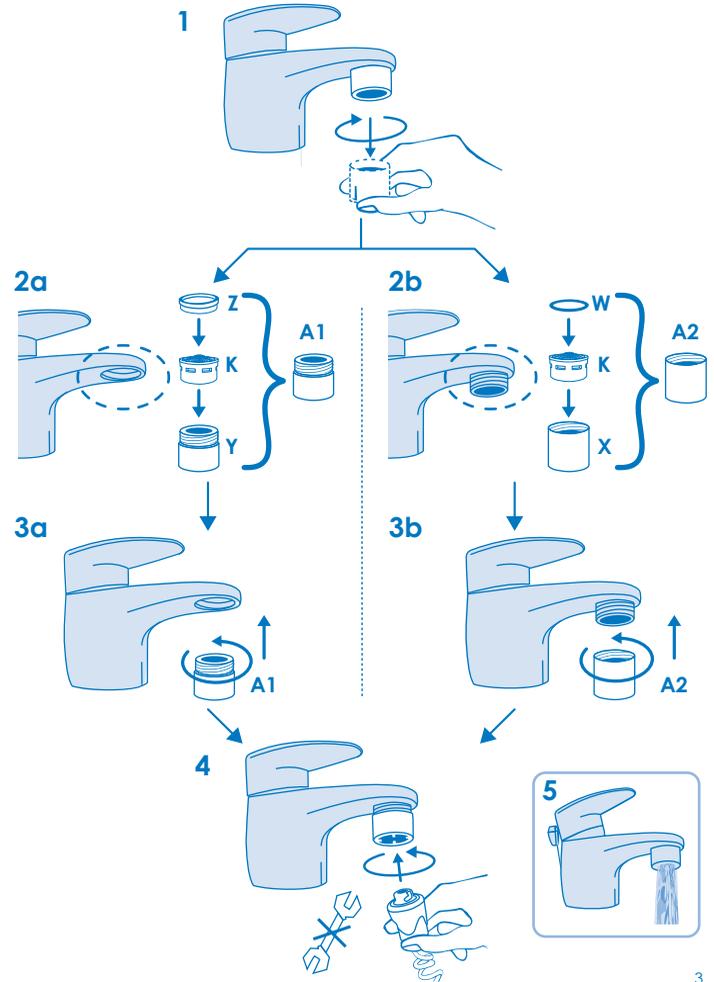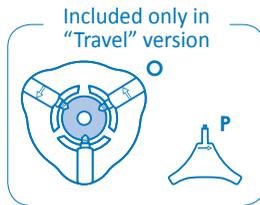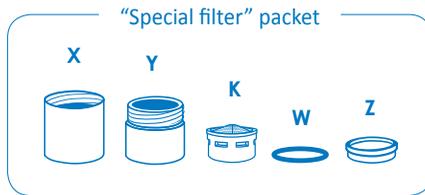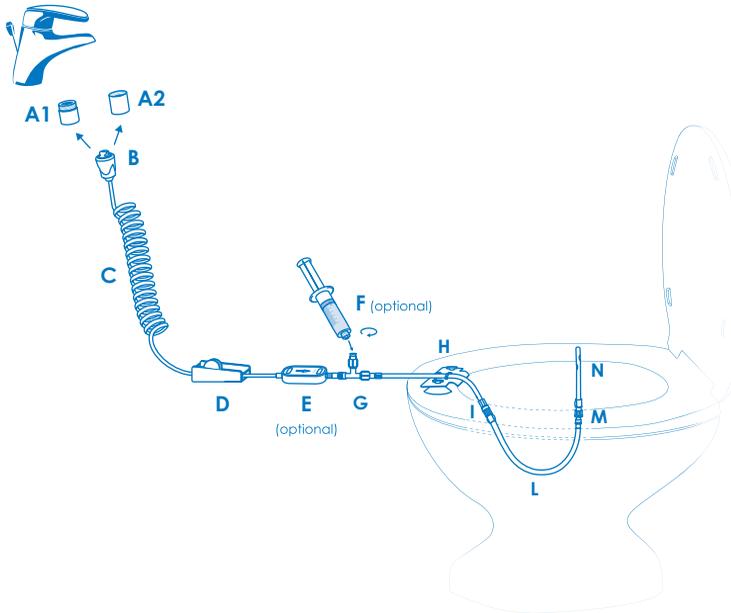


Attention: for "Basic" version go to ref. 15.


Addy Assembly

(From ref. 6 to ref. 14 only for "Travel" version -
for use at home see from ref. 1 to ref. 4)


6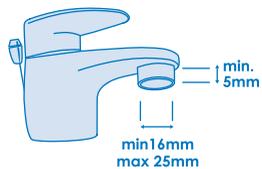


7

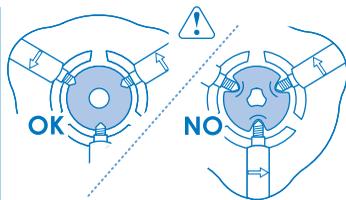
10

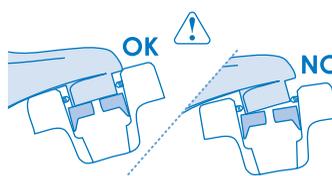
11

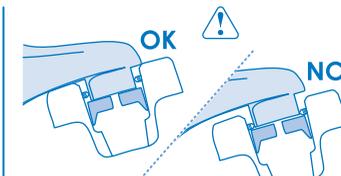
8

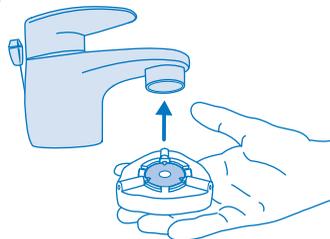
12

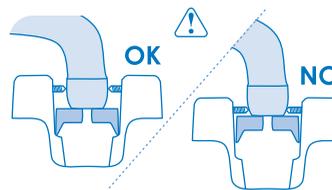
13

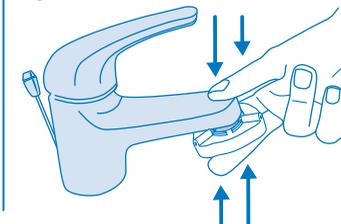
9

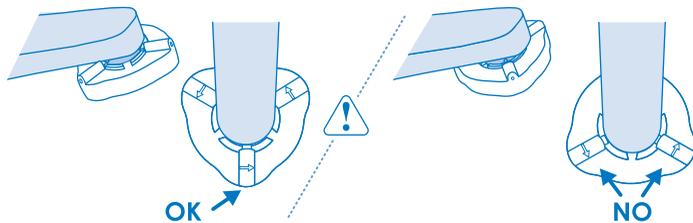
14

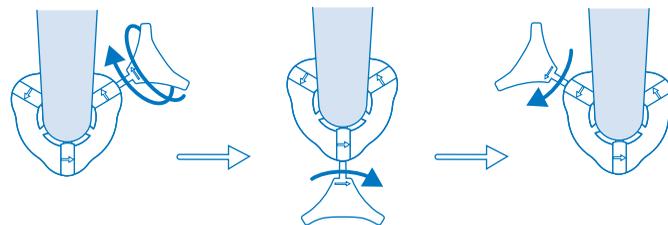
15

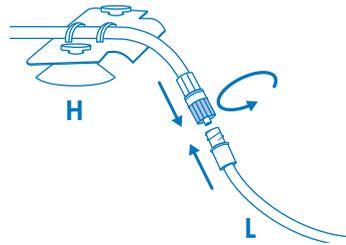16

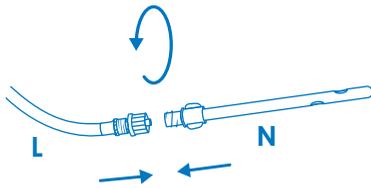
17

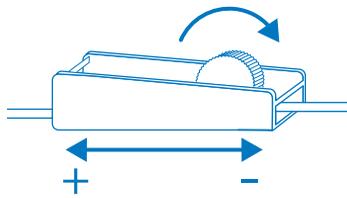
18a


18b Only for "Travel" version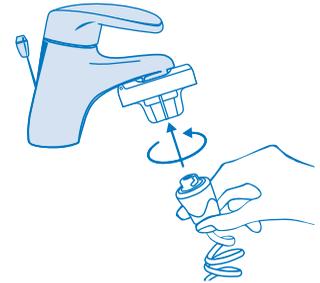


19

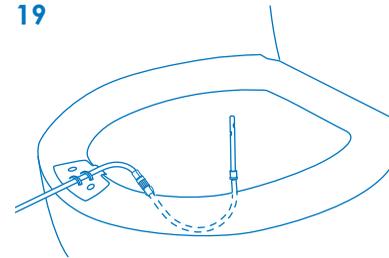
20

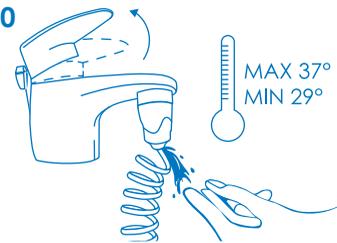
21

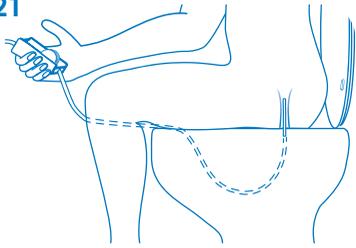22

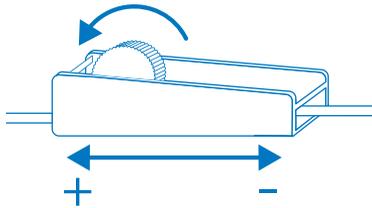
23

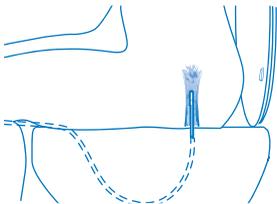24

25 Only for "Travel" version

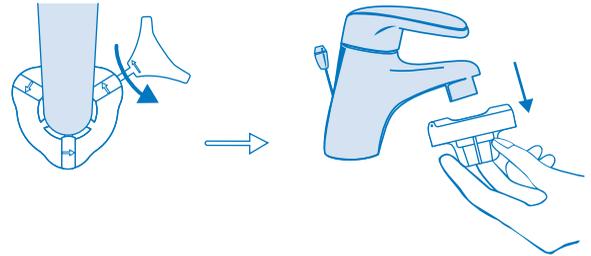
Maintenance

26

27

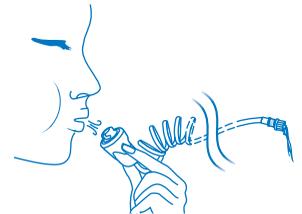THE PACKAGE CONTENTS:

- **Packet "Special Filter" containing:**
 - Special Attachment with internal thread (X)
 - Special Attachment with external thread (Y)
 - Aerator Filter (K)
 - Gasket (W - with circular section) for Special Attachment with internal thread
 - Gasket (Z - with rectangular section) for Special Attachment with external thread
- **Packet n. 1 containing:**
 - Connector (B) with 3mt spiral tube (C) and Manual flow regulator (D)
 - Filter for water with double membranes of 1.2 micron (E - accessory/optional)
 - Syringe (F - accessory/optional)
 - Connector for Phyto Regula (G) with tube with support and suction caps (H) and coloured mobile fitting (I)
- **Packet n. 2 containing:**
 - Semi-rigid modelling tube (L) with one-way check valve (M)
- **Packet with Rectal Pipes containing:** n. 12 disposable rectal pipes (Rif. N)
- **Quick Universal Adaptor "Addy" (O)** with fastening key (P)****
- **My Perfect Colon case**
- **User Guide**

Letters in parenthesis refer to general pattern

** Components E and F are not contained inside the basic package and they can be purchased separately (F is included in the package of Phyto Regula solutions).*

***only included with the "Travel" package.*

WHAT IS MY PERFECT COLON

My Perfect Colon is a medical device for home colon wash and cleanse (sigma and rectum).

WHY USE MY PERFECT COLON

The My Perfect Colon cleanse softens and helps the removal of fecal residue from the colon walls. My Perfect Colon is especially recommended for constipation, bloating, flatulence, for general intestinal problems and to maintain personal rectal hygiene. It is also recommended for people confined to bed, for those who are immobilized and/or have fecalomas.

CONTRAINDICATIONS

Use of My Perfect Colon is not recommended in case of: Enteritis-colon proctitis (Inflammation of the small and large intestine and of the anus); Acute abdominal pathologies; Pregnancy; Bleeding hemorrhoids and anal rhagades; Perforation of the rectum; Rectum cancer; Hematochezia; Melena; Recent colon, rectum

or anus surgery; Heart failure/ respiratory failure; Epilepsy and psychosomatic syndromes; Diarrhea; Dismetabolic Syndromes; Artery-venous pathologies; Diverticular disease in active phase; Nausea or vomiting; Serious state of dehydration; Prostate surgery.

In addition to the pathologies listed above requiring special precautions, it is always advisable to seek medical advice from your doctor before using this medical device.

WARNINGS – PRECAUTIONS OF USE

- In case of discomfort after cleansing, do not use My Perfect Colon again and seek medical advice from your doctor;
- Water temperature: My Perfect Colon must be used with warm water. Water temperature should be maintained between 29° and 37.5° C. After connecting My Perfect Colon to the tap, and dosing the flow regulator (D), check the temperature of the water coming out of the "regulatory pressure" valve (B) with your hands (fig. 20). Keep in mind that the rectal duct's sensitivity to hot or cold is higher than the hands. If during use you feel that the temperature is too hot or too cold, shut off the water flow immediately and regulate the water temperature again using the tap. Since some models of gas boilers don't start heating the water right away, it is possible that after connecting My Perfect Colon to the tap the water doesn't come out warm: in this case, turn on another tap at the same time to allow the boiler to start.
- Rectal pipe (N): you are advised to insert the rectal pipe no more than 4-5 cm (checking the two holes from which the water flow comes out are inserted well inside the anal duct). The rectal pipe is disposable; it must be used for one wash only, even if it is used by the same person.
- Water purity: the water used for the colon cleanse must be clean (normal drinkable water supplied by the domestic water network). If you believe that the water coming out of your tap is not of good quality or is not drinkable, we recommend using a My Perfect Colon medical filter with double membrane of 1.2 microns (E) to do the colon cleansing.
- Water flow: My Perfect Colon is the only at home colon cleanse system (for rectum and sigma) with a valve for automatic control of water flow (B - patented system), for guaranteed safety. The water flow in excess is automatically drained into the sink (fig. 24). Water flow can also be controlled manually during the wash, thanks to the flow regulator (D). It is possible hearing a sound similar to a whistle when the flow regulator is completely closed off and the tap is still running; this is normal and it's due to

the high pressure created inside the valve of the connector. It's possible to eliminate this sound by reducing the water flow from the handle of the tap.

- Colon cleansing must be always done with a low water flow, especially for the first washes, regulating the flow with the handle of the tap and the manual flow regulator (D). In case of diverticulosis, the water flow must be set to minimum.
- Semi-rigid modeling tube (L): model the tube curving it, making sure you do not fold or crush it, to ensure right water flow.
- Use by children: the My Perfect Colon medical device can be used by children only with the pediatrician's approval, and always under the supervision of one parent.
- After each colon cleansing, it is useful to take milk enzymes in order to reintegrate the intestinal bacterial flora.
- It can sometimes happen that the effect of washing is so great that the stimulus may return several times over the following 2-3 hours. It is recommended, especially for the first cleanses, to perform the colon cleansing only when you have the possibility to use a restroom over the following 2-3 hours. Should this happen, we suggest gradually reducing the quantity of water used.

ASSEMBLY**Installation of the filter to the tap***

To connect the My Perfect Colon special filter (packet "Special Filter") to the tap (sink, bidet, shower hose** or bath):

- 1 - Unscrew the filter of your tap, making sure the gasket of your old filter inside the tap is removed (fig. 1).
- 2a - If your tap has an internal thread (fig. 2a), insert the Aerator Filter (K) inside the Attachment with external thread (Y) and lay the gasket with rectangular section (Z) on the Aerator Filter (fig. 2a). Take the Filter with external thread (A1) you just assembled and screw it on the tap (fig. 3a);
- 2b - If your tap has an external thread (fig. 2b), insert the Aerator Filter (K) and the gasket with rectangular section (W) inside the Attachment with internal thread (X) (fig. 2b). Take the Filter with internal thread (A2) you just assembled and screw it on the tap (fig. 3b);
- 3 - Screw the new filter to the tap until it stops using the connector (B – fig. 4) with dry hands, with counterclockwise rotation.

* The installation of the filter on the tap must be done only the first time you use My Perfect Colon:

the exclusive My Perfect Colon filter should not be removed and installed again each time, as the tap can be used normally (fig. 5), ensuring aeration and water filtration. (The My Perfect Colon special filter is adaptable to all taps according to the CEN – EN 246 norm. Generally, this standard is respected by all taps on the market with screwable filter).

** To connect My Perfect Colon to the shower hose use the multi-function joint (accessory can be purchased separately - see the "Spare Parts and Accessories" section).

My Perfect Colon can be connected to the tap also via universal adapter "Addy" (O - Included only with My Perfect Colon Travel; you can purchase it separately for My Perfect Colon Basic - see the "Spare Parts and Accessories" section) without unscrewing the filter, ideal while travelling.

Installation of the My Perfect Colon device

- Screw the connection of the semi-rigid modeling tube (L - packet 2) to the blue mobile joint (I), placed at the end of the tube with support and suction caps (H) (fig. 15).
- Screw a rectal pipe (N) onto the check valve (M) placed at the end of the semi-rigid modeling tube (L - fig. 16).

QUICK UNIVERSAL ADAPTOR "ADDY"

Included only with My Perfect Colon Travel; you can purchase it separately for My Perfect Colon Basic. Addy (O) allows you to connect My Perfect Colon to the tap in a quick and easy way, without having to mount the threaded filter (A1 / A2).

It is ideal for use while travelling, in a hotel or in case of non-standard tap thread.

Addy can be connected to all the taps with water filter with an outside diameter from 16mm to 25mm and minimum length of 5 mm (fig.6).

ASSEMBLY to the TAP

Check that the three screws are loose and the gasket is properly in place, without visible deformation (fig.7). Hold Addy as in Figure 8, pushing up until the gasket completely adheres to the filter holder.

Place Addy under the tap making sure that:

- one of the 3 screws is aligned with the tap and the two remaining screws are on the side, making sure that none of the screws is under the neck of the tap (fig.9);
- it follows the inclination of the filter holder of the tap to allow the gasket to adhere completely and ensure the seal (fig. 10);
- it is as high as possible compared to the filter holder of the tap (fig. 11).

Special Cases

- Some filter holders have a slight nook: please make sure that the tips of the screws do not lean against the nook to prevent Addy from detaching.
- In case of taps that are slightly flared / rounded (like kitchen taps), mount the screws on the straight side of the filter and never on flared parts to prevent the adaptor from coming off the tap (fig. 12).

Holding Addy to the tap with the aid of your thumb as shown in fig. 13, use the fastening key (P) to screw the three screws to a secure hold on the tap, following the direction indicated by the arrow positioned on the cylindrical housing of the screws (fig. 14). Once you tighten one of the screws all the way, it is recommended for even stronger hold, to tighten the remaining two screws, without forcing them (fig. 14). While tightening the screws, be careful not to damage the inner gasket to avoid compromising the hold (fig. 7).

INSTRUCTIONS and SUGGESTION FOR ADDY:

- Gradually open the tap, do not open the water flow all at once. For a correct use of My Perfect Colon it is sufficient to open the tap not at full pressure, but just until the water flows from the valve located in the quick connector.
- Initially, for the first few moments, there could be some small leakage of water from the upper part of Addy but this will quickly stop. If the leakage persists or becomes excessive, it is recommended to repeat the process of hooking Addy to the tap.
- Don't touch or move Addy with opened water flow from the tap, in order to avoid compromising the seal.
- When you are done with Addy, simply unscrew one of the three screws to remove it from the tap (fig. 25).
- After using Addy it is recommended to dry it and leave the screws slightly tightened making sure, however, that the tips of screws do not deform the gasket (fig.7).
- If you do not use Addy for a long time, it is possible that upon the first re-use the movement of the screws may be harder: without mounting Addy to the tap, try to unscrew and tighten the screws several times to unlock and facilitate the movement without force. Should the screws not unlock, use some spray lubricant in the screw holes where you would insert the fastening key.
- The use of Addy in case of excessive pressure during the screwing and repeated use on the same tap can cause light marks on the filter holder of the tap due

to contact with the screws.

- If you leave Addy mounted on the tap, for each new use is recommended to check that the hold is solid, making sure the 3 screws are well tightened.
- When you unscrew the screws, do not force them once they've reached the end of the travel.
- For extended use of My Perfect Colon on the same tap, it is recommended to use special filters with threaded adaptors.
- Make sure the filter holder is screwed well on the tap to prevent leakage once the adaptor is mounted.

HOW TO USE MY PERFECT COLON

1. Completely close the flow regulator (D) placed at the end of the spiral tube moving the milled wheel towards the tube with support and suction caps (fig. 17 – position “.”).
2. Connect My Perfect Colon to the tap filter (A) or Addy (O) using the quick connector (B), inserting and rotating counterclockwise (fig. 18a – 18b) – for a better hold, connect the quick connector using dry hands.
3. Attach the tube with support and suction caps (H) on the frontal side of the toilet seat or on the toilet itself, pressing on the two suction caps. The rectal pipe (N) must be oriented upwards, in the center of the toilet and the semi-rigid tube (L) modeled to a “U” shape (as in the “General Pattern” and in fig. 19).
4. Turn on the tap and regulate the desired temperature (we recommend warm – max 37.5 °C), checking the temperature of the water coming out of the regulatory pressure valve with your hands (B - fig. 20 - inside the quick connector, there is a water flow regulatory valve – patented system - which will automatically flow all the water directly into the sink/bidet, as the water passage through the device has been closed previously by the flow regulator – D - point 1).
5. Open slightly the flow regulator (D) to check that water is coming out of the rectal pipe, all the connections are correct and there are no leakages.
6. Keeping the tap turned on, close the flow regulator (D) to interrupt the passage of water inside the device.
7. Sit on the toilet, placing the tube with support and suction caps (H) between the legs and, after adjusting the position of the semi-rigid modeling tube (L), insert the rectal pipe (N) gently into the anal duct up to about 4-5 cm, assuring that the two holes from which water comes out are inserted well inside (fig. 21).

8. Once seated, gradually open the water passage using the flow regulator (D – fig. 22) up to the desired level and then start the colon cleanse (fig. 23 - see the sections “Frequency and Duration” and “Colon Cleansing Advice” for instructions for use during the cleanse).
9. Once the cleanse has finished, remain seated on the toilet and close the passage of the water using the flow regulator (D – fig. 22); gently ease out the rectal pipe (N).
10. Turn off the tap and unscrew the rectal pipe (N) from the semi-rigid modeling tube (L - reverse operation to fig. 16). Do not use the same rectal pipe again for other washes. The rectal pipe is disposable and it must not be used again, even by the same person.
11. Wash the semi-rigid modeling tube (L – fig.26): unscrew the blue mobile fitting (I – reverse operation to fig. 15), remove the final part of the semi-rigid tube and clean it carefully, rinsing it thoroughly. Then dry the semi-rigid modeling tube. We suggest cleaning all the other parts regularly as well.
12. Remove the quick connector (B) from the tap and the support (H) from the toilet seat. Put the dry My Perfect Colon back into its case.

FREQUENCY AND DURATION OF WASHES

The My Perfect Colon cleanse is done for personal well being: we suggest it takes place in a relaxed environment, taking the necessary time and with no rush.

- It is advisable to do no more than one wash per week for the first month and no more than 3 washes per week afterwards, unless instructed otherwise by your doctor;
- Each wash consists of these 3 phases:
 - 1) Filling (flow of the water into the rectum);
 - 2) Holding (water is held inside the colon);
 - 3) Emptying (evacuation).
- For a correct execution of the cleanse it is recommended to alternate phases of filling with phases of holding, each of the duration of 5-10 seconds;
- For the first few washes, repeat the filling and holding twice. For further washes, repeat the steps five times. At any point, stop filling with water immediately if you feel full, even if this is the first filling and performed only for a couple of seconds;
- The total amount of water used during the cleanse is subjective and it depends on the personal body structure and condition. Normally, for each cleanse, you should not insert more than 1,5 lt of water overall, during the whole cleanse (1,5 lt passes

through the pipe on average in about 40 seconds with the tap and water flow fully open.);

- With the last phase of water filling, try to hold water inside (without excessive effort and for no longer than 4-5 minutes) before evacuating. After a few uses, the resistance to the stimulus will be higher;
- When you let water in the rectum, start the water flow slowly;
- The total duration of the cleanse may vary and it is subject to, other than the time necessary for the filling, how long you can hold the water inside the colon and the time necessary to evacuate, which can vary from case to case.

COLON CLEANSING ADVICES

- It is preferable, in case of high sensitivity, to grease the rectal pipe (N) with Vaseline, oil or similar product before use, in order to facilitate the insertion of the pipe;
- Before stating the cleanse, check the integrity of the device and correct assembly of the pipes;
- Model the semi-rigid tube (L) until it forms a “U” shape (see the General Pattern and fig. 19): put the rectal pipe (N) upwards inside the toilet;
- To facilitate evacuation, take a deep breath, contract the stomach and massage the area above the pubis and the left side of the abdomen for about 1-2 minutes. The evacuation will come naturally when the colon is ready and the feces are softened;
- Sensitivity to pressure and water temperature are subjective. Each user should evaluate and regulate them in accordance with their needs, always keeping in mind all contraindications (see the section);
- We recommend, especially during the initial cleanses, using My Perfect Colon with a moderate water flow (using the manual flow regulator). In case of excessive water flow, reduce it using the flow regulator;
- During evacuation, the rectal pipe (N) may be left in or taken out of the anal duct;
- Before the cleanse finishes, expel all water from the colon. If all water is not expelled, subsequent light leakage or slight discomfort may occur.

MAINTENANCE AND CONSERVATION

- We recommend keeping the product well cleaned and dry in its case between cleanses;
- The final part of the semi-rigid modeling tube (L) must be washed after each cleanse (fig. 16), prior to being put back in its case, and the connection should be removed (point 11 section “How to use My Perfect Colon”). We also recommend cleaning

all other parts of the My Perfect Colon device every time it is used;

- Do not use aggressive cleansers for cleaning the product because they could damage plastic or rubber parts;
- Before using the product, connect My Perfect Colon to the tap and let the water flow through the device and out of the pipe for about 30 seconds
- At the end of each cleanse, blow into the quick connector (B) to remove all residual water inside the tubes, in order to keep My Perfect Colon in hygienic condition (fig. 27);
- Between uses, it is advisable to keep the product with the flow regulator (D) open (fig. 22, position “+”).

SPARE PARTS AND ACCESSORIES

It is possible to purchase the complete My Perfect Colon product line on our website: www.mypfectcolon.com.

Rectal pipes: are available in refill packages of several units.

Double Membrane Water Filter of 1.2 microns: to filter bacteria and impurities of more than 1.2 microns from the water.

Board: it allows to do the My Perfect Colon cleanse lying down, thereby increasing the effectiveness of the cleanse, as the water can enter more deeply into the intestine. The board is placed on the toilet, and thanks to its hole and drainage area, all fecal residues go directly into the toilet, allowing evacuation without interruption of the cleanse. Made from high-strength polyethylene, the board is water-proof and easy to wash.

Concentrated solutions “Phyto Regula”: to add through connector for Phyto Regula into the My Perfect Colon device, in order to increase the effectiveness of the cleanse. Available in several formulations and suitable for various intestinal problems. The package of Phyto Regula solutions contains the dispenser syringe.

Universal Adapter “Addy”: allows My Perfect Colon use without unscrewing the tap filter. It is ideal while travelling, in hotels or in case of non-standard tap threads.

Multi-function joint for shower tube: to connect My Perfect Colon to the shower tube.

My Perfect Colon filter in metal: available with standard internal or external thread.

Adapters: for taps with non-standard threads.

Extension: 3 meter spiral tube with flow regulator, to connect My Perfect Colon in case the distance from the tap exceeds the 3 meter of the hose already

provided.

All other components of the My Perfect Colon device are also available separately: Spiral tube (3mt) with quick connector; Connector and tube with support and suction caps; Semi-rigid modeling tube.

The manufacturer, distributors and retailers refuse any liability for any damage that may, directly or indirectly, occur to people or property from the non-compliance with all requirements specified in this manual and that regard installation, use, contraindications and maintenance of this device.

My Perfect Colon is not and does not substitute equipment used in hydro-colon therapy centers. For any clarification or information please visit www.mypfectcolon.com or contact customer care via email at: customer@cpsrl.net.

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CSP S.r.l.
CSP S.r.l. uses only solar power.